medi GmbH & Co. KG
Medicusstraße 1
D-95448 Bayreuth
Germany
T +49 921 912-0
F +49 921 912-780
ortho@medi.de
www.medi.de

medi Austria GmbH
 Adamgasse 16/7
 6020 Innsbruck
 Austria
 T +43 512 57 95 15
 F +43 512 57 95 15 45
 vertrieb@medi-austria.at
 www.medi-austria.at

medi Belgium bvba
 Posthoornstraat 13/1
 3582 Koersel
 Belgium
 T +32 011 24 25 60
 F +32 011 24 25 64
 info@medibelgium.be
 www.medibelgium.be

medi Brasil
 Rua Neusa 216
 Diadema-Sao Paulo
 Cep 09941-640
 Brazil
 T +55 11 3201 1188
 F +55 11 3201 1185
 sac@medibrasil.com
 www.medibrasil.com

medi Danmark ApS
 Vejlegardsvej 59
 2665 Vallensbæk Strand
 Denmark
 T +45 46 55 75 69
 F +45 70 25 56 20
 kundeservice@sw.dk
 www.meditandmark.dk

medi Bayreuth España SL
 C/Canigó 2 – 6 bajos
 Hospitalet de Llobregat
 08901 Barcelona
 Spain
 T +34 93 260 04 00
 F +34 93 260 23 14
 medi@mediespana.com
 www.mediespana.com

medi France
 Z.I. Charles de Gaulle
 25, rue Henri Farman
 93297 Tremblay en France Cedex
 France
 T +33 1 48 61 76 10
 F +33 1 49 63 33 05
 infos@medi-france.com
 www.medi-france.com

medi Hungary Kft.
 Bokor u. 21.
 1037 Budapest
 Hungary
 T +36 1371 0090
 F +36 1371 0091
 info@medi-hungary.hu
 www.medi-hungary.hu

medi Medical Support Sdn Bhd
 medi representative office Asia
 Unit No. B-2-19, Block B, No.2,
 Jalan PJU 1A/7A
 Oasis Ara Damansara, PJU 1A,
 47301 PETALING JAYA
 Selangor Darul Ehsan
 Malaysia
 T: +6 03 7832 3591
 F: +6 03 78323921
 info@medi-asia.com
 www.medi-asia.com

medi Middle East
 P. O. Box: 109307
 Abu Dhabi
 United Arab Emirates
 T +971 2 6429201
 F +971 2 6429070
 info@mediuae.ae
 www.mediuae.ae

medi Nederland BV
 Heusing 5
 4817 ZB Breda
 The Netherlands
 T +31 76 57 22 555
 F +31 76 57 22 565
 info@medi.nl
 www.medi.nl

medi Polska Sp. z o. o.
 Zygmunta Starego 26
 44-100 Gliwice
 Poland
 T +48 32 230 60 21
 F +48 32 202 87 56
 info@medi-polska.pl
 www.medi-polska.pl

medi Bayreuth Unipessoal, Lda
 Rua Eugénio de Castro Rodrigues, nº2
 1700-184 Lisboa
 Portugal
 T +351 21 843 71 60
 F +351 21 847 08 33
 medi.portugal@medibayreuth.pt
 www.medi.pt

medi RUS LRC
 Business Park "Rumyantsevo"
 Rumyantsevo, Build. 1
 Leninsky Township
 142784 Moscow Region
 Russia
 T +7 495 229 04 58
 F +7 495 229 04 58
 info@medirus.ru
 www.medirus.ru

medi UK Ltd.
 Plough Lane
 Hereford HR4 0EL
 United Kingdom
 T +44 1432 37 35 00
 F +44 1432 37 35 10
 enquiries@mediuk.co.uk
 www.mediuk.co.uk

4 047872 707954

F011286 / 02.2014

medi Hip one

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Important notes

This brace is made for single patient use only. If it is used for treating more than one patient, the manufacturer's product liability according to the Medical Devices Act will become invalid. If undue pain or an unpleasant sensation develops while you are wearing the support, please consult your doctor or orthotist immediately. Do not wear the brace over open wounds, and use it only as instructed by your doctor or orthotist.

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medi Hip one

Intended Purpose

The medi Hip one is a functional hip orthosis. The product is exclusively to be used for the orthotic fitting of the hip and only on intact skin.

Indications

- For conventional treatment and dislocation of the hip
- Traumatic hip joint dislocations
- Acetabular fracture
- For postoperative implantation (e.g. after TEP)
- Loosening of the hip prosthesis
- After implantation of a hip joint prosthesis (TEP exchange)
- Femoral head resection

Contraindications

None known at present.

Fitting instructions

Please apply orthosis to patient in a standing position. First, open all belts and adjust the joint to the desired range of motion (see "Joint adjustment"). Then close the pelvic restraint. In order to adjust the hip piece to specific dimensions, you can use the four outer screws for dorsal adjustment (see "Adjustment of pelvic restraint") as well as the abdominal binder.

Now, adjust the thigh cuff. Make sure that the thigh cuff is centred over the medial femoral condyle and that the joint is at the approx. height of the hip pivot point. Close the thigh cuff belts. Now, adjust the adduction or abduction angles following the physician's instructions (see "Adjustment of adduction and abduction"). Figure 3 shows a correctly applied Hip one. Then verify that all screws have been tightened. This should be done in regular intervals.

Adjustment of pelvic restraint

Loosening the four dorsal screws enables the adjustment of the pelvic restraint width within the elongated holes. Once again, tighten all screws.

Positioning the joint

Loosen the screws of the joint support splint around the pelvis and thighs. Now you can adjust the joint height. Once the joint is correctly positioned, tighten the screws.

Adjustment of adduction and abduction

In order to adjust the joint adduction and abduction you must loosen the connecting screw between the joint and the thigh joint support splint. Each groove corresponds to 10°. Once you have found the desired position, tighten the screw.

Adjustment of flexion and extension

In order to adjust flexion or extension you must loosen the screw to take off the cap. Now you can remove or shift pins. The holes are at an angle of 15°. Once the desired range of motion is adjusted, reposition the cap and tighten the screws.

Care instructions

Soap residues, lotions and ointments can cause skin irritation and material wear.

- Wash the paddings by hand, preferably using a medi clean detergent, or in delicate cycle at 30°C using a mild detergent without fabric conditioners.
- Do not bleach.
- Leave to dry naturally.
- Do not iron.
- Do not dry clean.

Storage instructions

Keep the product in a cool, dry place and do not expose to direct sunlight.

Material composition

The frame is made out of aluminium, the belts out of velour and the belt ends out of polyester. The upholstery consists of PU foam.

Warranty / contractual guarantee

Consistent with legal guidelines.

Disposal

The product can be disposed of in the domestic waste.

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